# Polimid

**Sutura chirurgica sterile in poliammide plurifilamento rivestita. Non assorbibile.**  
**Sutura quirúrgica estéril en poliamida polifilamento revestida. No absorbible.**

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### Descrizione

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## ISTRUCTION FOR USE

### Description

**POLIMID** suture is a sterile surgical suture device of synthetic origin, composed of polyamide 6. **POLIMID** suture is formed by a core of twisted multifilaments coated with a sheath of the same material. It is available both dyed black (Palamid Black c.i.15710), in a range of gauze sizes (USP/EP) and lengths, attached to stainless steel needles of different sizes and shapes, as further described in the general catalog.

**POLIMID** is considered to be a pseudomonofilament suture since the fine filaments of its core give smoothness and resistance to the suture while the external sheath makes it comparable to monofilaments'. As for most of monofilament sutures, **POLIMID** is indicated to suture infected and/or contaminated tissue. In addition, thanks to its lack of adherence to tissues, it is used as a pull-out suture in post-operative course. The suture is not absorbed nor subject to degradation by the action of enzymes and it maintains its own tensile strength in "vivo". As with any foreign body, it may elicit a slight inflammatory reaction followed by encapsulation of the suture from fibrous connective tissue.

### Indications

**POLIMID** sutures are intended for use in all surgical procedures, at the user's discretion.

### Contraindications/Precautions/Reccomendations

**POLIMID** sutures are to be used by qualified personnel experienced in technical and surgical procedures where the use of non-absorbable sutures is required. Care should be taken when handling the suture material to avoid any bending or flattening to the thread due to the application of surgical instruments such as forceps or need holders. Greater knot security requires standard surgical technique of flat and square knots, to be applied with additional throws as may be indicated by the surgical circumstances and the surgeon's experience. Surgical sutures are disposable medical devices and must be disposed of as hospital waste material.

### Adverse reaction

As any suture material, **POLIMID** is a foreign body and may cause inflammatory reaction associated to erythema formation.

### Content

36 sterile envelopes.

### Warnings

Keep in cool dry place, away from any direct heat.

Expiration date is shown on each single envelope and on the outer box.

Disposable.

Do not use if package is damaged.

### Sterility

**POLIMID** sutures are sterilized by Gamma Ray.

Sterile unless package is opened or damaged.

Do not resterilize.

Reserved for the exclusive use by surgical medical personnel, in surgery or in hospital setting.

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Sweden & Martina S.p.A.