Polisoft

Sutura chirurgica in poliestere plurifilamento intrecciata e rivestita in silicone (non assorbibile).
Sutura quirúrgica en poliéster polifilamento trenzada y revestida en silicona (no absorbible).

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Descrizione

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ISTRUCTION FOR USE

Description

POLISOFT suture is a sterile non-absorbable surgical suture device of synthetic origin. The basic element of the suture thread is a macromolecule deriving from the polycondensation of terephthalic acid with ethylene glycol. **POLISOFT** suture is available both dyed green (D&C Green N°6 - C.I. 61565) and it is available in a range of gauge sizes (USP/EP) and lengths, attached to stainless steel needles of different size and shapes.

POLISOFT suture has excellent biological tolerance and causes a minimal initial inflammatory reaction in the tissues, followed by encapsulation of the suture in the fibrous connective tissue. Further details are described in the general catalog. **POLISOFT** suture meets the requirements of the Pharmacopoeia European Union for Braided Sterile Non-Absorbable Synthetic Sutures.

Indications

POLISOFT sutures are intended for use in all surgical procedures, at the user's discretion.

Contraindications

There are no particular contraindications.

Precautions

POLISOFT sutures are to be used by qualified personnel experienced in technical and surgical procedures where the use of non-absorbable sutures is required. Care should be taken when handling the suture material to avoid any bending or flattening to the thread due to the application of surgical instruments such as forceps or needle holders.

Greater knot security requires standard surgical technique of flat and square knots, to be applied with additional throws as may be indicated by the surgical circumstances and the surgeon's experience. Surgical sutures are disposable medical devices and must be disposed of as hospital waste material.

Adverse reaction

As any suture material, **POLISOFT** is a foreign body and may cause inflammatory reaction associated to erythema formation.

Content

36 sterile envelopes.

Warnings

Keep in cool dry place, away from any direct heat.

Expiration date is shown on each single envelope and on the outer box.

Disposable.

Do not use if package is damaged.

Sterility

POLISOFT sutures are sterilized by Gamma Ray.

Sterile unless package is opened or damaged.

Do not resterilize.

Reserved for the exclusive use by surgical medical personnel, in surgery or in hospital setting.

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Sweden & Martina S.p.A.